

Quick Start Process Guide

Sales Representative Portal Access

Contact a TRUMATCH® Solutions Case Coordinator for credentials.

New Image Center Validation

New Image Centers will need to implement a test scan for validation of requirements using the TRUMATCH Personalized Solutions Protocol (DS Huddle).

- i. Test system using Sawbones (sawbones.com, part #1107-1) following the TRUMATCH Personalized Solutions Protocol (DS Huddle).
- ii. Send images to DePuy Synthes or upload electronically. Note: keep the raw data on the imaging machine for at least 15 business days, as a scan may require reformatting.

Imaging team evaluates the scan and sends notification of a pass or fail. If failed, the scan will need to be reformatted using the existing raw data (if available) or by testing a new scan. Once passed, the center is ready.

New Surgeon Registration and Credentials

Log in to trumatchsolutions.com, click **Register Surgeon Registration Form** and Submit. A username and password are sent to the surgeon by a TRUMATCH Solutions Case Coordinator. Our team of Case Coordinators will review, approve, and build a custom profile based on the surgeon's surgical preferences.

Scanning and Sharing Studies

Scanned studies can be electronically uploaded or can be shipped overnight:

Electronically: <http://trumatch.secureimagesend.com/>

Mailing: TRUMATCH Personalized Solutions
DePuy Orthopaedics, Inc.
700 Orthopaedic Drive
Warsaw, IN 46581-0988

Proposal Design and Approval

Surgeon will receive and review the patient proposal via surgeon portal with 4 options:

1. **Approve:** guides are sent to manufacturing.
2. **Approve with noted changes:** proposal is updated with surgeon changes and guides are then sent to manufacturing.
3. **Redesign and resubmit:** proposal is updated and then sent back for surgeon approval.
4. **Cancel:** case is cancelled.

Upon approval, delivery date is given.

Manufacturing/Shipping

Guides are 3D-printed and shipped sterilized two weeks following approval.*

Shelf Life Guidelines

Guides are good for 180 days from the date of manufacturing (date is found on the box label) and cannot be resterilized if expired or dropped. If any of these happen, we can rebuild the case utilizing the existing patient image.**

TRUMATCH Solutions 3.0 Code Listings

Femoral/Tibia Cut or Pin	
Part #	Description
420578	TRUMATCH CT PIN GUIDE KIT L
420579	TRUMATCH CT PIN GUIDE KIT R
420580	TRUMATCH CT PIN GUIDE FEM L
420581	TRUMATCH CT PIN GUIDE FEM R
420582	TRUMATCH CT PIN GUIDE TIBIAL L
420583	TRUMATCH CT PIN GUIDE TIBIAL R
420915	TRUMATCH CT CUT GUIDE KIT L
420916	TRUMATCH CT CUT GUIDE KIT R
420917	TRUMATCH CT CUT GUIDE FEM L
420918	TRUMATCH CT CUT GUIDE FEM R
420919	TRUMATCH CT CUT GUIDE TIBIAL L
420920	TRUMATCH CT CUT GUIDE TIBIAL R

Femoral Cut Guide/Tibia Pin Guide	
Part #	Description
420906	TRUMATCH CT FEM CUT TIB PIN L
420907	TRUMATCH CT FEM CUT TIB PIN R

Contacting a Case Coordinator

TRUMATCH Case Coordination Team

1-800-689-0746

trumatchsupport@its.jnj.com

Office hours are M-F, 7:30-4:30 p.m. EST

*Is an estimation only. Shipping timeline may be extended based on changes requested during approval.

**Will need surgeon's approval if images are more than 4 months old.