

DERMABOND® PRINEO® Skin Closure System

Postoperative Care Sheet

DERMABOND PRINEO System combines DERMABOND® Topical Skin Adhesive with a self-adhering mesh (latex free) for closure of surgical incisions and simple lacerations.



DERMABOND PRINEO System forms a strong, flexible, waterproof microbial barrier that falls off naturally after 7-14 days.

Removing the system prematurely can disrupt the healing process.

Postoperative Dressings

- Additional dressings are not needed with DERMABOND PRINEO System
- Dry dressing (like gauze) may be applied after the DERMABOND is no longer tacky to the touch. A compression garment or other covering may also be used at the surgeon's discretion
- If using another type of dressing, the sticky portion should not be placed over the DERMABOND PRINEO System, as it may pull it off when removed

Showering

- Patients may shower right away if directed by the healthcare professional
- After showering, the wound area should be gently blotted with a soft towel. It should not be soaked or scrubbed
- Patients should not swim or otherwise immerse the wound for prolonged periods

