

DERMABOND Advanced®

How to Care for Your Wound After It Is Treated with DERMABOND ADVANCED® Topical Skin Adhesive¹


DERMABOND ADVANCED Adhesive is a sterile, liquid skin adhesive that holds wound edges together. The film will usually remain in place for 5 to 10 days, then naturally fall off your skin. The following will answer some of your questions and provide instructions for proper care for your wound while it is healing.

Check Wound Appearance

- Some swelling, redness, and pain are common with all wounds and normally will go away as the wound heals. If swelling, redness, or pain increases, or if the wound feels warm to the touch, contact a doctor. Also contact a doctor if the wound edges reopen or separate

Caring for Bandages

- If your wound is bandaged, keep the bandage dry
- When changing the dressing, do not place tape directly over the DERMABOND ADVANCED Adhesive, because removing the tape later may also remove the film

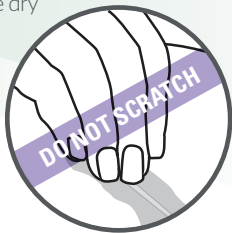

Avoid Topical Medications

- Do not apply liquid or ointment medications or any other product to your wound while the DERMABOND ADVANCED Adhesive is in place. These may loosen the film before your wound is healed

Keep Wound Dry and Protected

- Apply a clean, dry bandage over the wound if necessary to protect it
- You may occasionally and briefly wet your wound in the shower or bath. Do not soak or scrub your wound, do not swim, and avoid periods of heavy perspiration until the DERMABOND ADVANCED Adhesive has naturally fallen off
- After showering or bathing, gently blot your wound dry with a soft towel. If a protective dressing is being used, apply a fresh, dry bandage, keeping the tape off the DERMABOND ADVANCED Adhesive
- Protect your wound from injury until the skin has had sufficient time to heal
- Do not scratch, rub, or pick at the DERMABOND ADVANCED Adhesive. This may loosen the film before your wound is healed
- Protect the wound from prolonged exposure to sunlight or tanning lamps while the film is in place

If you have any questions or concerns about this product, please consult your doctor.

Reference:

1. DERMABOND ADVANCED® Topical Skin Adhesive Instructions for Use. Ethicon, Inc.