

# HARMONIC<sup>®</sup> Shears

## Assembly Guide

### Assembly

While holding the Hand Piece in a **vertical orientation**, attach the Hand Piece to the instrument by rotating the instrument onto the Hand Piece in a clockwise rotation (finger tight only).



Use the torque wrench (already mounted to the shaft) to tighten the instrument onto the Hand Piece. Turn the torque wrench clockwise while holding **only** the gray Hand Piece until it clicks twice.



- Note: Do not use any other means than the torque wrench to attach or detach the instrument from the hand piece.
- Note: Do not torque the instrument by hand without the torque wrench.
- Note: Hold only the gray Hand Piece handle while applying the torque wrench.



### Pre-Run Testing



Connect the assembled Hand Piece and instrument to the generator and turn the generator power on. Follow On-Screen Instructions to complete initiation.

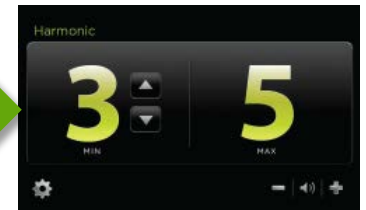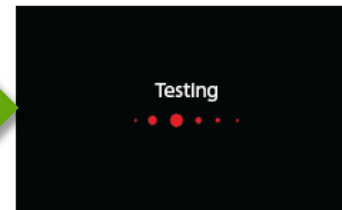

The test prompt screen appears when the initiation sequence is complete. Activate the instrument with the jaws open for 2 seconds to run test until the HARMONIC<sup>®</sup> ready screen is displayed.

**For all product details, including a complete list of warnings and precautions, please reference the Instructions for Use and Generator Operator's Manual.**